

Our pan-range injection system with a unique audible indicator



### Welcome to the Click™

- Patented Haptic Control System**  
Increases delivery consistency in C loop\* lens profiles
- Pre-Loaded**  
No handling of the lens
- Micro Incision**  
2mm tip for micro incision
- OVD Free**  
Self-lubricating chamber
- Multi Product Platform**  
Compatible with numerous Lenstec™ products
- Double Click™ Indicator**  
Provides audible reassurance of successful preparation
- Easy Glide Injection**  
Compressible injector tip to aid smooth delivery of lens
- Slow Release Spring**  
Provides a controlled and consistent delivery

### Advancing lens delivery in to a new era...

Lenstec™ has taken lens injection beyond the pre-loaded generation, using patented innovation to develop the Click™ system; a pan-range platform to introduce multiple lens designs through the same injection system.

The Click™ Haptic Control System allows delivery of C-Loop\* lenses with increased consistency whilst the Easy Glide Tip comfortably condenses when transitioning through the injector, to deliver a smooth consistent motion.

\* The patented control system is not necessary for plate haptic design IOLs




## Click™ Pan-Range Injection System Guide




STEP 1

Initial click of the double click indicator

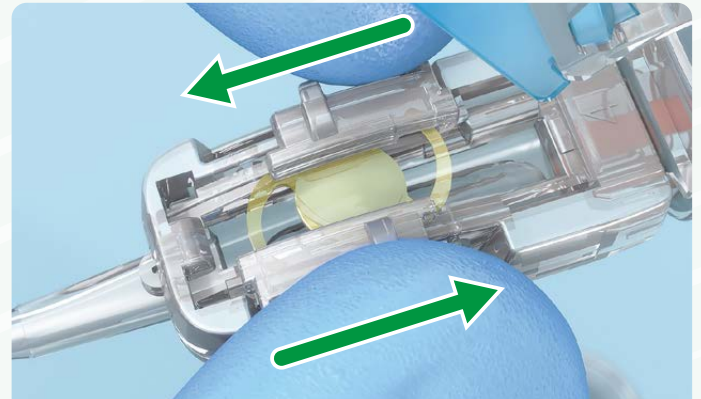

STEP 2

Patented haptic control system  
Applies to C loop\* lenses only

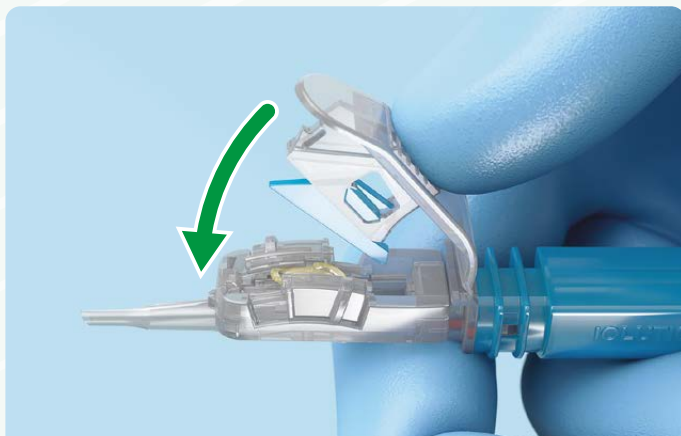

STEP 3

Second click indicates lens folding process is complete



STEP 4

Resulting in smooth delivery of the lens

## Completing the injection process

To complete the injection process, the slow-release spring provides controlled and consistent delivery of the lens ensuring its soft tip does not protrude from the cartridge. The Click™ provides zero handling of the implant, with a one-system-fits all platform, reducing the need for theatres to hold multiple products at the same time, eliminating potential errors using the incorrect lens/injector combination. With Click™ it's possible to totally regulate the procedure using the same knife, hand piece, incision, and injector for all compatible lenses.

The Click™ injection system is compatible with the following intraocular lenses...



Get in touch with Lenstec™

World Headquarters  
 lenstec.com  
 lenstec@lenstec.com

Manufacturing  
 Lenstec (Barbados) Inc  
 lenstecbarbados@lenstec.com

UK Sales  
 lenstecuk.com  
 lenstecuk@lenstec.com

PKB90 Rev 1

EU Rep  
 CMI, Bratislava  
 lenstec@cmi.sk